


Attends® Cover-Dri



Fitting Guide

Preparation


1

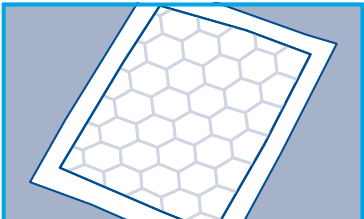

Open the product fully making sure that the waterproof back sheet is positioned on the bed and the soft white absorbent cushions are next to the skin.

Fitting

2

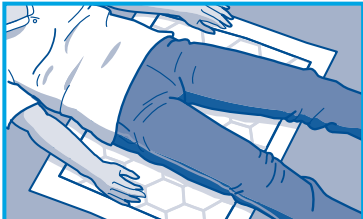

Position the person on the underpad.

Changing

3



Turn the person into the lateral position and roll up the used underpad towards the persons back.

4




Unfold the new underpad, positioning it towards the back of the person.

5



Turn the person on to the underpad and ensure that the pad is lying flat on the bed. Remove the used product.

6

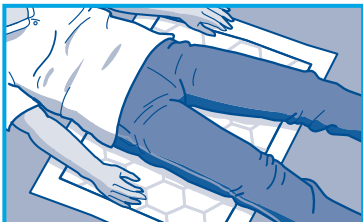

Lay the person in their original position.

Hints and Tips

Selection

- Choose the correct product size and absorbency dependent on the clinical situation you are using the product in.

Leakage

- Check that you have the correct size and absorbency of underpad for the situation it is being used in.

Disposal

- Roll up pads and dispose as per your Trust's clinical waste policy.
- Do not flush the pad down the toilet.



Attends Cover-Dri are tested and approved by proDERM, Institute for Applied Dermatological Research.

For more information visit www.attends.co.uk